

For queries about the Audiomed Communicator
or to obtain more batteries please contact the
Audiology department on:

01782 674727

**The AudioMed communicator should be used as
a temporary solution to aid hearing only while
hearing aids are being prescribed or
cannot be worn.**

**A guide to using the
AudioMed communicator
to aid hearing loss.**



Inserting a battery

Pull down on the lip at the bottom of the control box. Insert the battery “flat side” facing upwards and close the drawer.

Using the controls

The listener has 3 controls – 0 for OFF, 1 for ON and 2 for TONE.

To turn the listener on, move the switch upwards.

Take care when increasing the volume to not cause loudness discomfort to the patient.

Once turned on, insert the earpieces into the ears and adjust them until it is comfortable for the patient.

Adjust the position of the tips in the ears until it is whistle-free.

Adjusting the volume

The volume wheel is located on the side of the box.

Rotating the wheel upwards increases the volume and downwards decreases the volume.

Remember to:

Turn the communicator off when it is not in use to preserve the battery life. Do this by moving the switch to the bottom position.

Clean the earpieces at regular intervals using a dry cloth.

If the communicator does not switch on, change the battery.